

NEXTSTEP ARTHROPEDEX

Reprocessing of Surgical Instruments – Offset Reamer handle


ASSEMBLING AND DISASSEMBLING PROCEDURE OF THE OFFSET REAMER HANDLE

DISASSEMBLING



Step 1

Fully assembled instrument

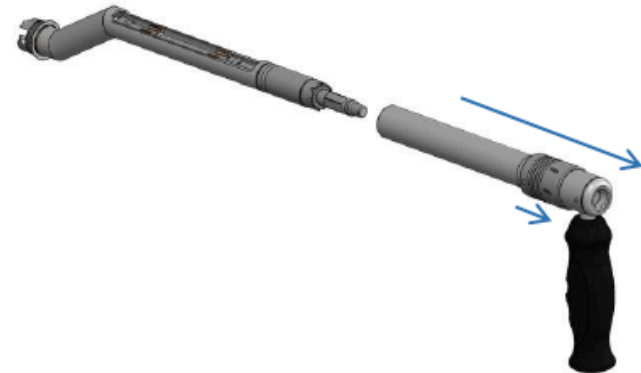

Step 2

Pull the sleeve backward to unlock and disassemble the handle

NEXTSTEP ARTHROPEDEX


Reprocessing of Surgical Instruments – Offset Reamer handle

DISASSEMBLING (CON'T)




Step 3

Pull the motor shaft out of the body

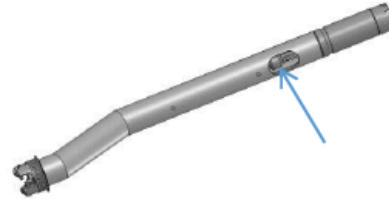

Step 4

Push the UJ chain at the back of the instrument and unsnap it the bearings

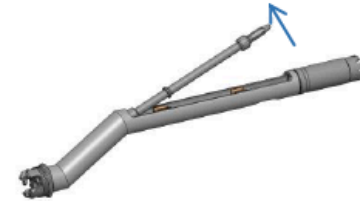

Step 5

Pull the UJ chain out of the body. Move the release sleeve of the head in the park position as indicated in page 4.



Step 6

Fully disassembled instrument. Cleaning and sterilization of the instrument shall be performed with the fully disassembled product.


NEXTSTEP ARTHROPEDIX

Reprocessing of Surgical Instruments – Offset Reamer handle


ASSEMBLING



Step 1
Fully disassembled instrument

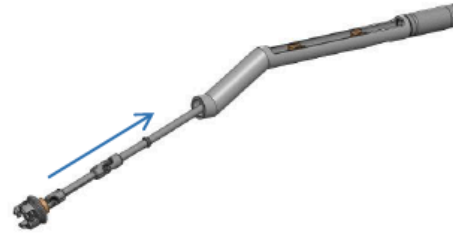


Step 2
Insert the UJ chain into the body

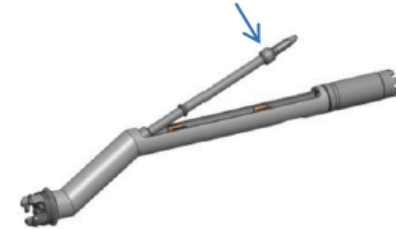

Step 3
Snap the shaft into the bearings



Step 4
Insert the motor shaft into the body

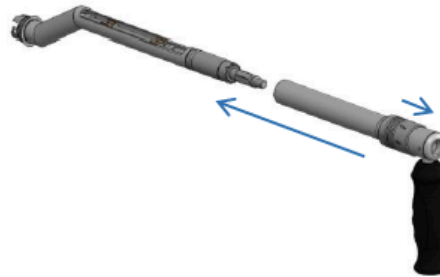

Step 5
Pull the sleeve backward to unlock and insert the handle of the instrument. Release the sleeve to lock




Step 6
Fully assembled instrument


NEXTSTEP ARTHROPEDIX

Reprocessing of Surgical Instruments – Offset Reamer handle

POSITION OF THE HEAD FOR CLEANING AND STERILIZATION

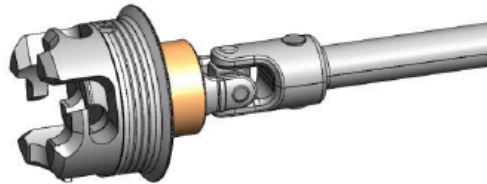


Position with the release sleeve forward

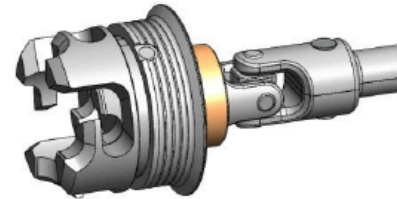


Position with the release sleeve backward for easy cleaning and sterilization

To change the position of the head

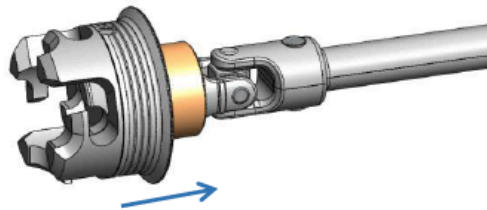

Step 1

Pull the release sleeve

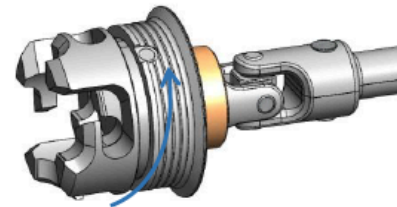

Step 2

Turn the release sleeve clockwise